




Tie-Up[™]

Canine Intravaginal A.I. Kit

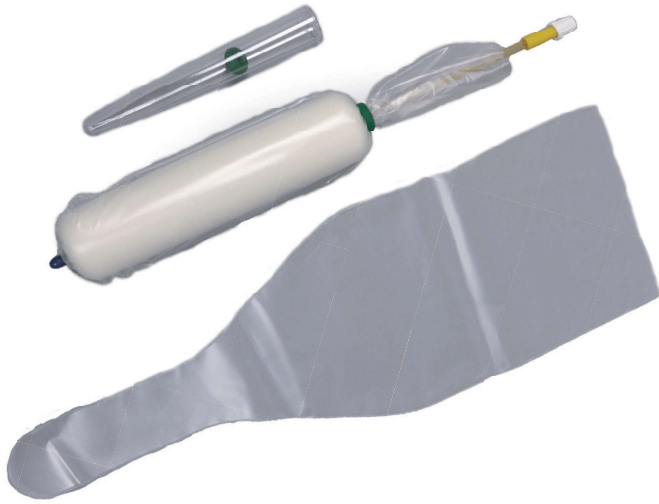

TIE-UP[™] CANINE INTRAVAGINAL A.I. KIT includes:

- Disposable Canine A/V Liner
- 15 ml Semen Collection Tube
- Canine Semen Delivery Device



INSTRUCTIONS FOR USE OF TIE-UP[™] CANINE INTRAVAGINAL A.I. KIT:


- 1) Cleanse vulva and external surrounding area with soap and warm water
- 2) Place collected semen from collection tube into a syringe
- 3) Remove the Canine Semen Delivery Device from sealed pouch; do not remove outer clear sheath that covers foam
- 4) Lubricate the device and the canine's vulva with non-spermicidal lubricant
- 5) Gently insert the Canine Semen Delivery Device into the vulva, STIP[™] Tip end first, leaving the capped Flex Tip Infusion Pipette end protruding
- 6) Push gently at an upward angle until the entire foam sponge area is inside the vulva, up to the green O-ring. The proper insertion position is with approximately 3" of the insemination tube and 2" of the clear plastic sheath protruding from the vulva. Once inserted properly, the device will not easily slip back out of the vagina as part of the "tying up" process
- 7) Remove cap from the insemination tube and attach the semen-filled syringe securely to the insemination tube
- 8) Slowly depress plunger of syringe to deliver entire contents into insemination tube
- 9) Remove syringe and immediately replace cap securely onto insemination tube to prevent loss of contents
- 10) Keep canine in a comfortable standing position for at least 30 minutes to simulate the tie up
- 11) Remove the insemination device by firmly grasping the end of the clear plastic sheath and pipette and gently pulling outward and discard device.



Tie-Up[™]

Canine Intravaginal A.I. Kit

Proper Insertion Diagram

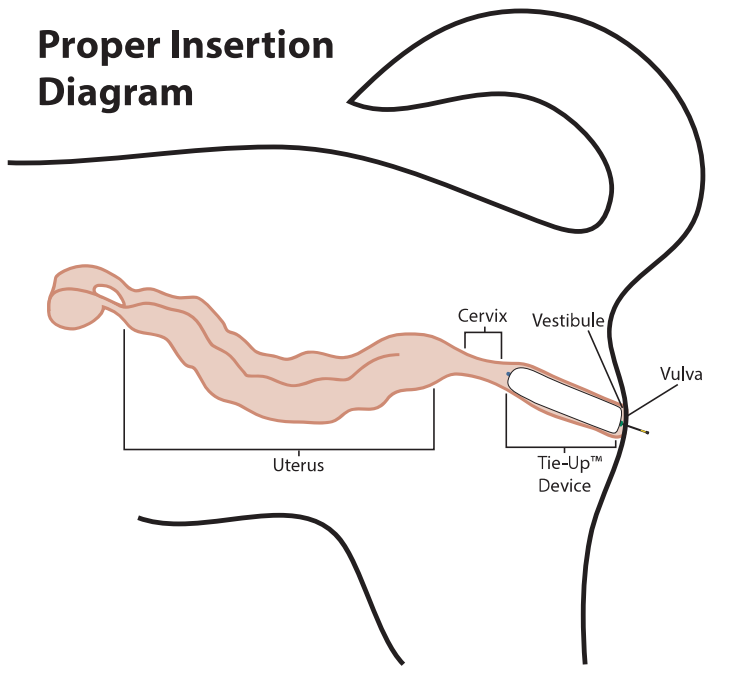

PROPER TIE-UP[™] DEVICE PLACEMENT:

Gently insert the Tie-Up[™] through the vulva and into the vagina until the green O-ring is just beyond the vestibular area. In most canines, this will place the blue STIP[™] Tip in close proximity to the external os (outer edge) of the cervix in the anterior vagina for semen delivery.

NOTES:

- If canine's hymen is present, continue to push device gently to penetrate and ease past the hymen. You will feel a slight "give" as the hymen is penetrated. During natural breeding, penetration of the hymen will cause sharp pain, but proper and gentle insertion of device will cause only slight momentary discomfort, if any.
- After the removal of the device, it is recommended that the animal be confined to a crate or small pen for an hour or more to encourage the dog to remain calm and to minimize stress.

Item	Product Description	UOM
18200-2	Tie-Up [™] Canine Intravaginal A.I. Kit, 2"	Each
18200-4	Tie-Up [™] Canine Intravaginal A.I. Kit, 4"	Each
18200-6	Tie-Up [™] Canine Intravaginal A.I. Kit, 6"	Each
18206-2	Tie-Up [™] Canine Intravaginal A.I. Device, 2" - Gamma Irradiated	5/Bag
18206-4	Tie-Up [™] Canine Intravaginal A.I. Device, 4" - Gamma Irradiated	5/Bag
18206-6	Tie-Up [™] Canine Intravaginal A.I. Device, 6" - Gamma Irradiated	5/Bag
18205-2	Tie-Up [™] Canine Intravaginal A.I. Device, 2" - Non-Sterile	5/Bag
18205-4	Tie-Up [™] Canine Intravaginal A.I. Device, 4" - Non-Sterile	5/Bag
18205-6	Tie-Up [™] Canine Intravaginal A.I. Device, 6" - Non-Sterile	5/Bag

MAI Animal Health - UNITED STATES
1.800.447.0687
sales@maianimalhealth.com

MAI Animal Health - EUROPE
+44 (0) 1225 791 184
sales.uk@maianimalhealth.com

MAI Animal Health - AUSTRALIA
+61 (0) 2 6372 6622
sales.au@maianimalhealth.com

MAI Animal Health - NEW ZEALAND
+64 (0) 9 444 2115
sales.nz@maianimalhealth.com